


Saliva Collection Kit Instructions


Kits with Straw and Saline Rinse Packet

See Public Health Ontario's [COVID-19 PCR Saliva Specimen Collection Instructions website](#) for more details on what to do before and after the specimen is collected.

Note: Saliva collection kits come may come with a straw of funnel and may also come either with or without a saline rinse packet, or a cap with blue transport liquid.

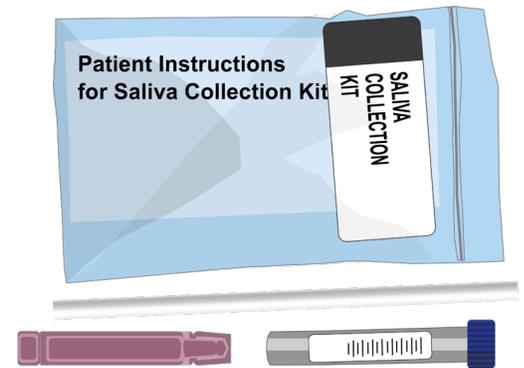


- A. Wash your hands with soap and water or hand sanitizer.
- B. Confirm kit contents (as shown in the top right image).

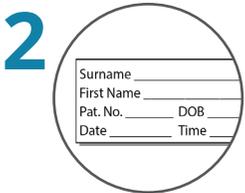

Make sure test tube is labelled with your:

- full name
- date of collection
- one other unique identifier (e.g., date of birth or Health Card Number)

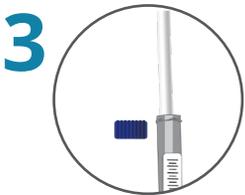

- A. Unscrew the tube cap and place cap upside down on a clean surface.
- B. Insert the straw into the tube.



- A. If masked, remove your mask.
- B. Open the packet of clear saline rinse provided.

NOTE: Use only the liquid provided in the packet.

- C. Squeeze all the saline rinse into your mouth.

NOTE: DO NOT SWALLOW.



- A. Swish saline rinse in your mouth for 5 seconds.
- B. Tip your head back and gargle for another 5 seconds.
- C. Repeat A & B for 5 seconds.
- D. Repeat A & B again for 5 seconds.



- A. Using the straw, slowly and gently spit the saline rinse through the straw directly into the tube.

Note: Take your time being careful not to splash or spill.

- B. Continue until the level in the tube reaches the fill line (around 2-3 ml), not including bubbles.

This may take 2-5 minutes.



- A. Remove and discard the straw.
- B. Screw the cap back on the tube.

Note: DO NOT reopen the tube or remove any liquid



- A. Check that the cap is screwed on tight.
- B. With a clean tissue, wipe the outside of the tube and cap.
- C. Check that label is complete (see Step 2).
- D. Place tube in clear biohazard bag, then place completed form in bag's outside pocket.



- A. Wash your hands with soap and water or hand sanitizer.
- B. If you were wearing a mask, put it back on.